

ULTRA SAFETY PLUS

TWIST



DO's & DON'Ts




Do not use the Ultra Safety Plus black handle




Use the blue Ultra Safety Plus Twist™ handle

Do not rotate the barrel of the syringe during the procedure as it might result in unlocking and separation

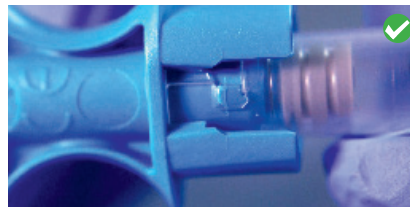


Lock the handle and the syringe together by twisting. Ensure that they stay locked throughout the whole procedure.

Do not twist the needle cap upon removal

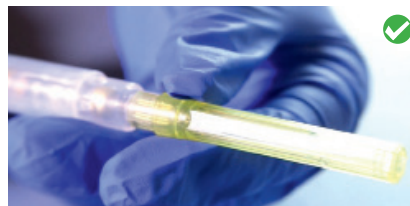

Before the injection, remove the needle cap by pulling it straight apart

Do not apply excessive pressure during the procedure

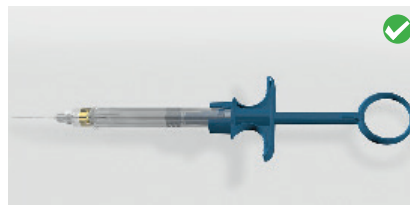

Towards the end of the cartridge, if you feel any resistance, stop the injection



Fully retract the plunger before separating the device

Do not bend the two components to separate



Rotate the barrel to **unlock the system** and carefully pull the two parts straight apart to separate

Ensure you use an USP Twist 'O' ring when replacing



VERY IMPORTANT
Ensure you replace the 'O' ring after 25 autoclaves and replace the blue handle after 100 autoclaves